

Thermoplastic Tape

Self-developed New product in domestic

Ideal for fixing small parts such as fingers and wrists



【 Scope of application 】

- ◆ Fracture and postoperative external fixation.
- ◆ Soft tissue, joint ligament tendon, nerve injury and postoperative external fixation.
- ◆ Postoperative external fixation of bone and joint deformity and postoperative external fixation of burns and plastic surgery.

【 Product Features 】


- ◆ New product in domestic, It has been researched and developed independently for several years, and now the new model is on the domestic market.
- ◆ After heating and softening, It can be shaped arbitrarily and fixed, which is more comfortable.
- ◆ Repeatable shaping, When the effect of shaping and fixing is not satisfied, It can be shaped and fixed twice after heating and softening.
- ◆ Easy to cut, it can be easily cut into required shapes and sizes according to the use part and strength.
- ◆ Comfortable and Breathable, Large mesh ventilation structure makes it comfortable to wear.
- ◆ Strength Customization, The strength can be customized by adjusting the number of layers.



Easy to use



Repeatable shaping

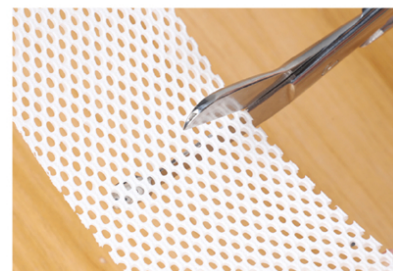

Easy to cut



Beijing Jinwei Kangda Medical Instrument Ltd.

Add: Room 1406-1409, 1st Building, No. 14 Courtyard, South Road of Western 3rd Ring-Road, Fengtai District, Beijing 100073, China

Tel: +86-10-63334994 / 63394999 Fax: +86-10-63334994

Web: www.kdbd.com.cn



【 How to use 】





- ◆ Wear the heat-insulating stockinette.
- ◆ Cut suitable size or shape into 80℃ hot water and heat for 1–3 minutes to soften it completely.
- ◆ Remove it with pliers and put it on a dry towel, squeeze out the water and dry it (be careful of burns).
- ◆ Fix the product on the limb and shape it with your fingers and the fish-bone area.

【 Cautions 】

- ◆ Use with caution on patients with open wounds, ulcers and infections on the affected area.
- ◆ Protect the skin with heat-insulating stockinette.
- ◆ The more layers, the stronger it is, and it can be reinforced by product layer stacking.
- ◆ Keep the position of the affected limb unchanged during the molding process until it is cured and formed
- ◆ Avoid close to flame and other heat sources or direct sunlight to prevent the product from softening.

【 Model specifications 】

Model	Specification	Color
2in	5.0cm × 50cm	White
2in	5.0cm × 200cm	White
3in	7.5cm × 50cm	White
3in	7.5cm × 200cm	White
4in	10.0cm × 200cm	White

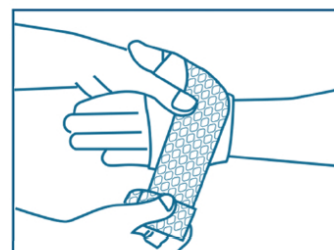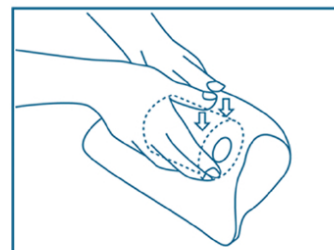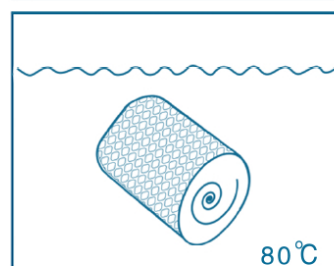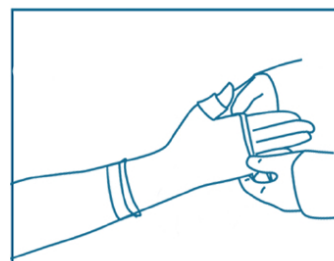

Beijing Jinwei Kangda Medical Instrument Ltd.

Add: Room 1406-1409, 1st Building, No. 14 Courtyard, South Road of Western
3rd Ring-Road, Fengtai District, Beijing 100073, China
Tel: +86-10-63334994 / 63394999 Fax: +86-10-63334994
Web: www.kdbd.com.cn

