



Jiujiang Huada Medical Dressing Co., Ltd

Add: Minjiashan Road Gongqing City Jiangxi Province

Tel: 86-792-4359088

Fax: 86-792-4341359

Description of product

1. Introduction of Product

Gauze rolls are manufactured and designed for the purpose, which used for cleaning trauma and absorbing blood of wound, insulating wound and preventing from infection. Primary raw material is degreased gauze. Gauze rolls, the sealed products which wrapped by paper bag, sterilized by radiation process and qualified by each performance examination, are supplied for usage of medical units or individual. Gauze rolls are in sterile state after radiation, and should destroy after using one time.

2. Use method and notice items

2.1 Use Operation method

Gauze rolls which apply to cleaning and absorbing blood in wound in various kind of surgical operation after sterilization.

- a、 Select suitable specification according to intended purpose of patient;
- b、 Check the integrality of package, confirm whether within the term of validity;
- c、 Wash hands clearly and put on medical glove, tear off wrapping paper, take out gauze;
- d、 Spread gauze in the wound, absorb gore or clean;
- e、 Observe the change situation of wound after operation.

2.2 Notice items

- a、 Sterile gauze rolls is limited to use only once immediately after opening the package, and should be destroyed after usage;
- b、 Usage of the products with damaged package are prohibited;
- c、 the storage should avoid high temperature, high wetness, and have non-corrosible gas and good aeration;
- d、 The product could be used after sterilization of radiation and qualified through various examination items;
- e、 The term of validity of sterile product is three years after production.

2.3 Contraindication

Before use, at first should clean hands and put on medical gloves, then spread gauze in the wound, prohibit to operating directly by hands, in order to avoiding infection in wound.

3. The intended usage of product

Gauze rolls are applied for the purpose, which used after sterilization for cleaning trauma and absorbing blood of wound, insulating wound and preventing from infection. Gauze rolls which are the basic devices of wound process in surgical operation apply to patients of various age.

4. Definition and classification of Product

The products are defined and classified according to the above-mentioned intended purpose and the classification standard of MDD93/42/EEC Annex IX

According to the classified standard of MDD93/42/EEC appendix IX, gauze rolls are passive, surgical non-invasive devices, which conform to stipulation of classification rules 4, they are in Class I*.

