

Instruction for Use

<Product: Surgical gown>



<Document No.: MG/CE-MDR-06-02-02>

<Date of issue: 2022.07.01>

Prepared by		Checked by		Approved by	
Name	Chengmingzhu	Name	Dingcunyan	Name	Xuweihua
Position	Editor Team	Position	Editor Team	Position	Approver
Date	2022.07.01	Date	2022.07.01	Date	2022.07.01
Signature		Signature		Signature	

Haian Medigauze Co., Ltd.

Device Description and Specification

Device Name	Specifications	Specification Length ×Width	Basic UDI-DI	Device figure
Surgical gowns (Standard)	S-1, M-1, L-1, XL-1, XXL-1, XXXL-1	110x130cm 115x135cm 120x140cm 125x145cm 130x150cm 150x160cm	697313440MGSG001 LT	
Surgical gowns (Reinforced)	S-2, M-2, L-2, XL-2, XXL-2, XXXL-2,	110x130cm 115x135cm 120x140cm 125x145cm 130x150cm 150x160cm	697313440MGSG002 LV	

[Caution]

- 1)Must be used within the specified period of validity, the package must be integral, no damage.
- 2)Damp or moldy products cannot be used.
- 3)Used only once, cannot repeat use.

[Use Method]

- 1) Take out sterile surgical gown. Wash hands and wear hats and masks before wearing surgical clothes. Take out the sterile clothes and wash your hands in a spacious place.
- 2) Shake off the surgical gown. Hold the collar of the surgical clothes with both hands and separate them to both sides. Shake off the surgical clothes with the cuffs.

3) Throw up the surgical gown, extend the arm into the cuff, gently throw up the surgical gown, extend the arm into the cuff, and extend the hand forward. Be careful not to pull it to the place with bacteria

4) put both hands forward and stretch out the cuffs, fasten the belt of the collar corner, and then with the assistance of the itinerant nurse, grasp the collar corner from behind and pull it back. The wearer will stretch out his hands forward and the nurse will fasten the belt of the collar corner.

[Storage conditions]

The product shall be stored in a dry, ventilated, non-corrosive environment and kept away from fire and flammable materials.

[Shelf-life]

The equipment can only be sterilized by EO, and the validity period of the sterilized product is 3 years

[Intended User]

Medical professionals in a specialty, clinical nurses

[Intended Patient Populations]

Medical staff and patients who need protection during the operation

[Intended using environment]

Hospitals, health centers, clinics.

[Side effect]

Contact infection

[Medical Conditions]

Need to block the spread of germs







[Indications]


















Operation room

[Contraindications]

It is forbidden for people who are allergic to non-woven fabrics

[Definitions of Signs]

Signs	Description	Signs	Description	Signs	Description
	Trademark of Manufacturer		Batch number		Date of Manufacture
	Medical device		Model number		Country of manufacturer

Signs	Description	Signs	Description	Signs	Description
	Use-by date		Sterilized using ethylene oxide		Caution
	Non-pyrogenic		Catalogue number		Unique Device Identifier
	Single sterile barrier system		Double sterile barrier system		Keep away from sunlight
	Do not reuse		Do not use if the package is damaged		Keep dry
	Do not resterilize		Authorized representative in the European Community		Manufacturer
	CE marking of conformity, and Notified Body Code		Consult instructions for use	/	/

Manufacturer

Name: Haian Medigauze Co.,Ltd.

Address: Jiaoxie Industry park, 226633Haian, Jiangsu, PEOPLE'S REPUBLIC OF CHINA

Tel: +86-513-88240958

Web: www.medigauze.com

E-mail: produce@medigauze.com

SRN: CN-MF-000009960

European Authorized Representative

Name: Shanghai International Holding Corp. GmbH (Europe)

Add: Eiffestrasse 80, 20537 Hamburg, Germany

E-mail: shholding@hotmail.com

SRN: DE-AR-000000001